


Quick Reference Guide – Lasix® ONYU (lay-six on-yoo) (furosemide injection) 80 mg/2.67 mL, for subcutaneous use with Infusor


Before You Start

- Do not start an infusion unless directed to do so by your healthcare provider.
- A healthcare provider should train you before using the Infusor by yourself.
- Start the infusion within 7 hours of preparing the Infusor.
- Do not use more than 1 Infusor at a time.
- Do not do more than 2 infusions in 24 hours.
- Be sure you are near a bathroom because you should urinate (pee) a lot during the infusion.


1 Charge Reusable Unit

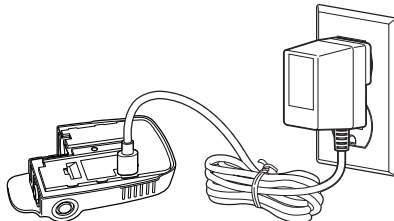

- Always charge the Reusable Unit before using a new Disposable Unit (before each infusion).
- Reusable Unit is charged** if you see a solid blue battery light:

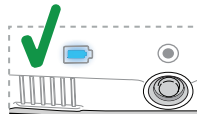


2 Check Expiration Date




- Wash hands** with soap and water.
- Check expiration date on the Lasix ONYU Kit.
- Do not** use if expired.

3 Open Lasix ONYU Kit

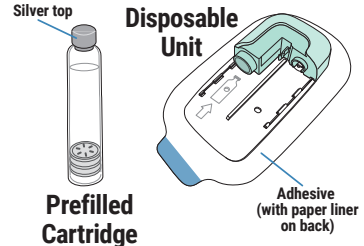


- Remove alcohol pads, Prefilled Cartridge, and Disposable Unit from the Lasix ONYU Kit.
- Do not** remove the paper liner from the back of the Disposable Unit.
- Do not** use if the Disposable Unit was dropped or if any parts appear to be broken. Use a new one.

4 Prepare Prefilled Cartridge

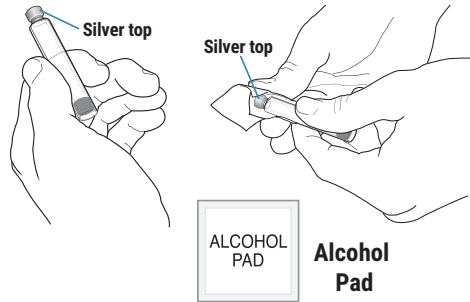

- Look at the Prefilled Cartridge.
- Do not** use if the medicine looks cloudy, yellow-brown, or you see floating particles.
- Wipe the silver top of the Prefilled Cartridge with an alcohol pad.

5 Insert Prefilled Cartridge

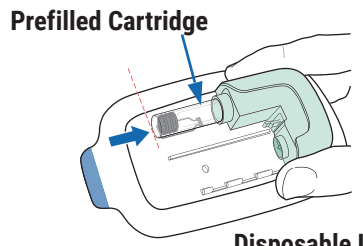


- Push the Prefilled Cartridge into Disposable Unit with the silver top inserted first.
- When fully inserted, the bottom of the Prefilled Cartridge should be at the tip of arrow on the Disposable Unit.

Warning


Warning: Do not use insulin cartridges or other medicine cartridges in the Infusor. Doing so could cause severe injury. Only use the Lasix ONYU Prefilled Cartridge with the Infusor.

Use this space to record your healthcare provider's phone number.


6 Slide Together

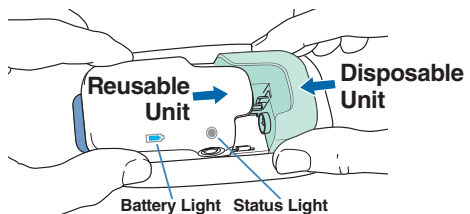

- Slide together until the Status Light on the Reusable Unit turns on.
- If Status Light is flashing red, go to Step 11 in the Instructions for Use.
- Infusor is ready** if you see flashing blue light and hear the OK Tone (1 long beep):




7 Choose Site

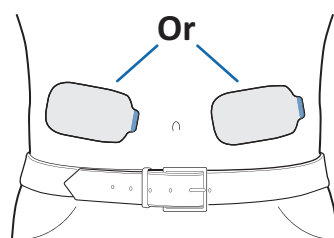


- Choose a stomach site above beltline, left **or** right of the belly button. Trim excessive hair.
- Do not** choose a site with irritated, broken, wet, or folded skin.
- Do not** wear clothing that may rub against or move the Infusor.

8 Wipe Skin

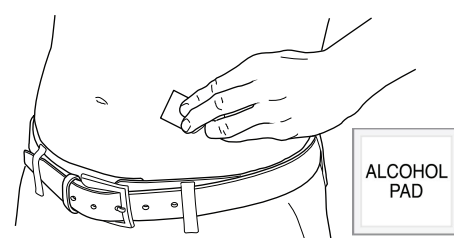


- Wipe the site with an alcohol pad.
- Let dry for at least 10 seconds.

9 Remove Paper Liner

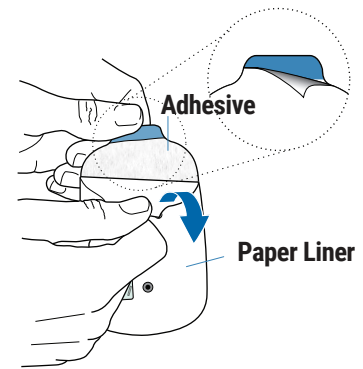


- Peel paper liner off adhesive on back of Disposable Unit.
- Only remove the paper liner. Leave the sticky adhesive on the Infusor.
- Do not** touch the sticky adhesive with your fingers.

10 Stick Infusor on Stomach

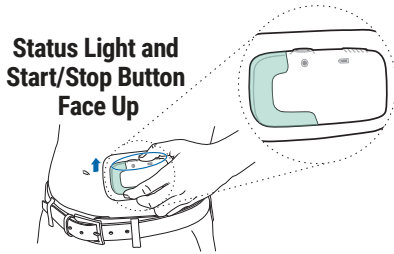

- Make sure the Status Light and Start/Stop Button are facing up in a horizontal position as shown, so you can see them when the Infusor is on your stomach.
- Rub the edges with a finger.
- Do not** put the Infusor on your stomach if you can see the needle. Go to **Troubleshooting: You Can See the Needle** in the Instructions for Use.

Manufactured for **SQ Innovation, Inc.**
20 Burlington Mall Road, Suite 220
Burlington, MA 01803
1-855-452-7496 (1-855-4LASIX6)
www.Lasix-ONYU.com




218594


Continue...


Quick Reference Guide – Lasix® ONYU (lay-six on-yoo) (furosemide injection) 80 mg/2.67 mL, for subcutaneous use with Infusor

11 Start Infusor	12 Wait 5 Hours	13 Check Status Light	14 Confirm Infusion
------------------	-----------------	-----------------------	---------------------

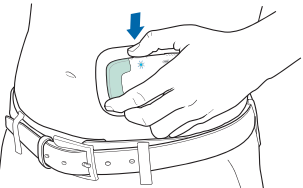


- Press and hold the Start/Stop Button until you hear the motor and see the blue Status Light flashing.
- You may feel a slight pinch.
- If the Status Light is flashing red, go to **Troubleshooting: The Status Light is Flashing Red** in the Instructions for Use.
- Infusor started** if you see flashing blue light and hear the OK Tone (1 long beep):

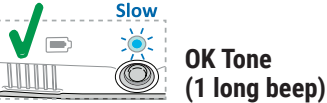




- Wait 5 hours for the Infusor to deliver the full infusion.
- You will **often hear the motor start and stop** during infusion.
- Do not** get the Infusor wet.
- Do not** remove the Infusor.

- After 5 hours, check the Status Light.
- If the Status Light is flashing red, go to **Troubleshooting: The Status Light is Flashing Red** in the Instructions for Use.
- Infusor is done** if you see solid blue light and hear the OK Tone (1 long beep):

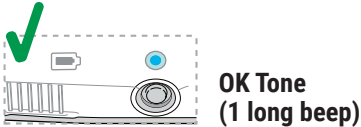

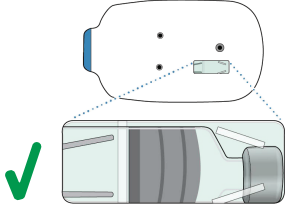


If **gray** stopper is in the middle, all of the medicine was delivered.


X If you see anything else, **call your healthcare provider right away.**

X


15 Confirm Medicine Worked	16 Pull Apart and Throw Away Disposable Unit	17 Clean the Reusable Unit	18 Store Product
----------------------------	--	----------------------------	------------------




- If you urinate a lot (at least 2 times) during the infusion, the medicine worked.
- If you urinate less than 2 times, the medicine may not have worked. **Call your healthcare provider right away.**

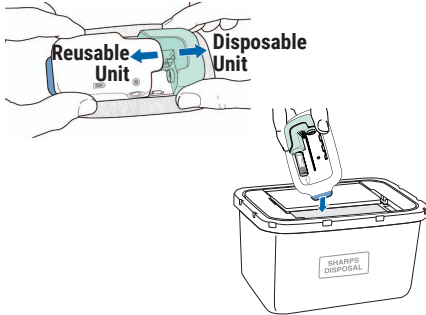

- Pull apart** the green Disposable Unit and white Reusable Unit
- Throw away** the **Disposable Unit with the Prefilled Cartridge** into an FDA-cleared **sharps disposal container**. You should not throw away the Disposable Unit in household trash.
- Do not** use a Disposable Unit or Prefilled Cartridge more than 1 time.

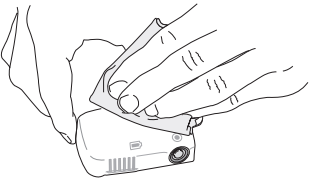

- Wipe** the white **Reusable Unit** with a clean, **water-damp cloth** to remove visible dirt. Let dry completely before storing.
- Do not** soak or spray the Reusable Unit with liquid.
- Do not** use cleaners with alcohol, bleach or other solvents. Mild detergents like dish soap are OK.



Make sure the Reusable Unit is completely dry before storing.


Store the Reusable Unit, Charger, and Lasix ONYU Kit boxes:

- At room temperature between 68°F and 77°F (20°C and 25°C).
- Do not** refrigerate or freeze
- Keep in a dark place to protect from light
- Keep dry
- Keep extra Prefilled Cartridges and Disposable Units in original package until it is time to use them.

Troubleshooting

See **Chapter 3: Troubleshooting** for more information.


Battery Light **flashing blue**

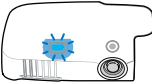

No

Battery is **still charging**. Wait until light is solid blue.


Status Light **flashing red and 2 short beeps**

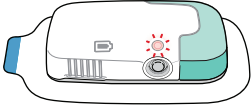

Alarm Tone

A **device or delivery error** occurred. Go to **Troubleshooting: The Status Light is Flashing Red** in the Instructions for Use.

Both Lights **solid red**

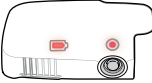

No

Reusable Unit is **expired** and cannot be used. Go to **Troubleshooting: The Reusable Unit Has Expired** in the Instructions for Use.

If you can **see the needle**:

- Do not** touch the needle because you could get an injury.
- Go to **Troubleshooting: You can See the Needle** in the Instructions for Use.

If you need to **stop the Infusor early** before the 5-hour infusion is done:

- Do not** remove the Infusor from the stomach before stopping it. Press and hold the Start/Stop Button until the status light flashes red and you hear the Alarm Tone (2 short beeps). The Infusion has stopped and the needle has pulled in.
- Remove the Infusor from the stomach.
- Pull apart the white Reusable Unit and the green Disposable Unit.
- Throw away the Disposable Unit with the Prefilled Cartridge into an FDA-cleared sharps disposal container.
- Call your healthcare provider** because you did not get all of the medicine.

See **Troubleshooting: Lights and Tones Table** in the Instructions for Use for additional troubleshooting help.